


CellEdge Trimming Blades and Handles


Manufactured from stainless steel, CellEdge Disposable Trimming Blades enable easy, non-deformational cutting of tissue.

The blades come in an easy-to-use dispenser which eliminates the need for each trimming blade to be individually wrapped in paper. This minimises the user's contact with the blade, protecting them from potential injury. The dispenser also incorporates a safety stopper to prevent the dropping of blades, and a handy disposal area at the bottom.

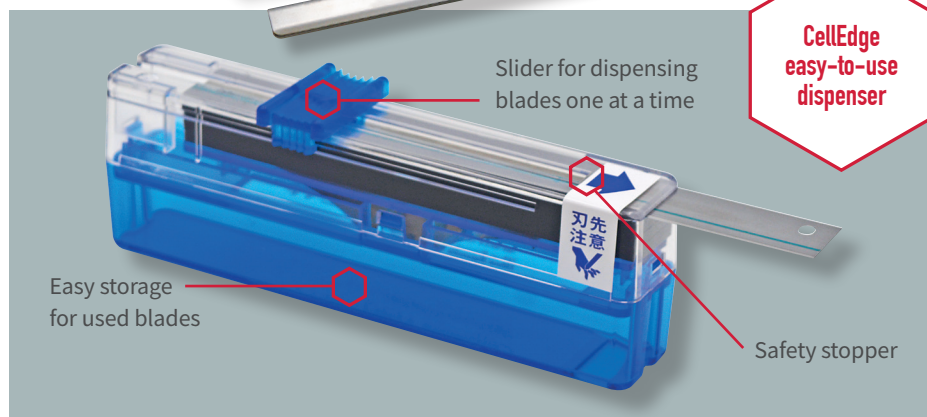


CellEdge Trimming Blades are easily set into our CellEdge Trimming Handles, designed specifically for this blade. These easy-grip handles are manufactured from durable plastic, making them easy to clean.

Item Code	Description	Quantity
CAH-0100-00A	CellEdge T130 Trimming Blade - 130mm, 50 Blades	1
CAH-0200-00A	CellEdge T260 Trimming Blade - 260mm, 50 Blades	1
CAH-0300-00A	CellEdge Trimming Handle for 130mm Blade	1
CAH-0400-00A	CellEdge Trimming Handle for 260mm Blade	1

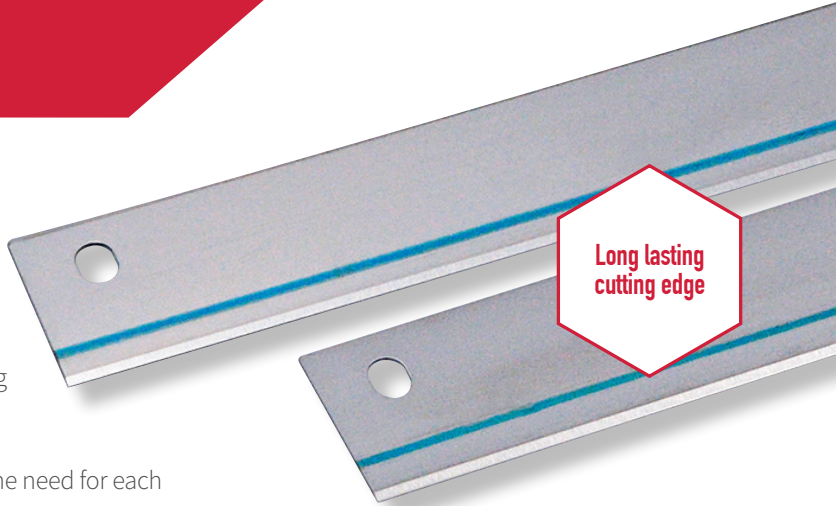

KEY BENEFITS

- 1 Easily cut a variety of tissue types – hard, soft, semi fixed
- 2 Dispenser offers improved user safety
- 3 High durability of blades and handles
- 4 Trimming Blades are easily set into trimming handles

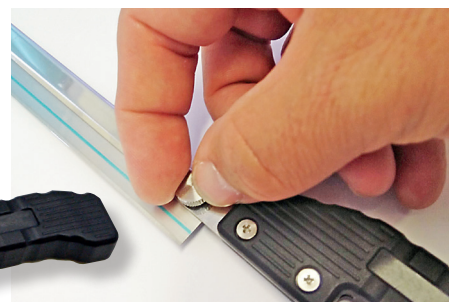

Instruction for Attaching CellEdge Trimming Blade to the Blade Handle

- Step 1:** Unscrew the locking screw all the way.
- Step 2:** Carefully slide the blade in from the top until the hole in the blade, and the screw hole align. The locking screw then needs to be tightened by hand.
- Step 3:** Assembly is now complete. Please use with care.



CellPath Ltd

80 Mochdre Enterprise Park
Newtown, Powys SY16 4LE UK
T: +44 (0)1686 611333
E: sales@cellpath.com
www.cellpath.com