# CellSafe1 Biopsy Cassette

EBG-03\*\*-03B, EBG-03\*\*-12A, EBG-03\*\*-99H,  
EBG-04\*\*-03B

Issue Date: FEB 2023 01.585.719 V4

CellPath Ltd  
80 Mochdre Enterprise Park,  
Newtown, Powys, SY16 4LE, UK  
T: +44 (0)1686 611333  
E: sales@cellpath.com  
cellpath.com

# CellSafe1 Biopsy Capsule

**EBG-03\*\*-03B, EBG-03\*\*-12A, EBG-03\*\*-99H, EBG-04\*\*-03B**

A small device intended to be used in the laboratory to support/contain clinical tissue specimens (e.g., biopsy samples) to facilitate specimen processing [e.g., fixation (encapsulation in paraffin wax), dehydration, infiltration] in preparation for subsequent cytological or histological examination, and storage. It is in the form of a synthetic polymer container, with or without a lid, with drainage holes to maximize fluid exposure during tissue submersion in a parent device into which the container is loaded. This is a single-use device.

For laboratory use.

Store in a dry place. Don't store in direct sunlight.

This product does not expire under normal environmental conditions.

Processing Limitations: Suitable for high temperature processing. Suitable for microwave processing.

Marking capabilities – EpreDia Printmate, Leica IPC, Sakura Autowrite, CellMark pens, 2B pencils.

## Instructions for Use

### STANDARD CASSETTE USE:

1. Insert the hopper tube into the EpreDia PrintMate printer.
1. Insert the thread into the Leica IPC printer. Holding the top cassette, pull the thread up to release.
1. Insert cassettes manually into printer loading system.
2. Place tissue into the CellSafe I Cassette. We recommend that only one biopsy of a maximum 4.5mm thick is placed in the cassette. Please ensure tissue is >400µm.
3. Using the lid provided, attach the lid. Close the lid and place the cassette in a fixative ensuring it is fully submerged.
4. Process the cassette through solvents according to your usual protocol until the specimens are ready for embedding. If you are using any unusual reagents, we suggest that you carry out preliminary chemical compatibility tests before using CellSafe I to contain tissue samples.

5. Carefully open the cassette.
6. Embed, cool and section as per your usual protocol.
7. Dispose of the lid in accordance with laboratory guidelines.

**DO NOT REUSE THIS PRODUCT.**

### BIOPSIES PLACED DIRECTLY IN CELLSAFE I:

1. Place tissue into the CellSafe I Cassette. We recommend that only one biopsy of a maximum 4.5mm thick is placed in the cassette. Please ensure tissue is >400µm.

**NOTE:** To prevent drying of the sample, it is recommended that a tiny amount of fixative is applied to the sample while samples are being collected.

2. Securely attached the lid provided. Close the lid and place the cassette in a fixative ensuring it is fully submerged.
3. Transfer the pot to the laboratory for processing, if applicable.

**NOTE:** The CellSafe I is x-ray transparent.

### STRAINING TISSUE FRAGMENTS:

1. Pre-soak the CellSafe I in ethanol or formaldehyde.
2. Ensure the fragments are >400µm.
3. Hold or support the cassette horizontally and slowly pour the suspension through the CellSafe I.
4. The tissue fragments will be retained in the CellSafe I, when all the suspension has been filtered; the CellSafe I can be closed.

**Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.**

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