Mounting Media for Coverslipping

SEA-1300-00A, SEA-1304-00A, SEA-1600-00A, SEA-1604-00A, SEA-1900-00A, SEA-1904-00A

ExPert XTF:

XTF Clearing Agent, 1-dodecanethiol, Isopar E, 2-ethylhexyl methacrylate, Isobutyl methacrylate

DPX:

Xylene

Expert:

Xylene, Toluene, Ethylbenzene

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

CellPath Ltd
80 Mochdre Enterprise Park,
Newtown, Powys, SY16 4LE, UK
T: +44 (0)1686 611333
E: sales@cellpath.com
cellpath.com

Mounting Mediums

SEA-1300-00A, SEA-1304-00A, SEA-1600-00A, SEA-1604-00A, SEA-1900-00A, SEA-1904-00A

A substance intended to be used to create a seal between a microscope slide, clinical specimen and/or coverslip to preserve the specimen and enhance image quality during microscopic examination.

Ready to use solution.

For laboratory use.

Keep in a cool, dry, well ventilated area. Keep containers tightly closed. Store in correctly labelled containers.

Please see expiry on product label.

Clearing Agent Compatibility: DPX: Xylene; ExPert: Xylene; ExPert XTF: XTF Clearing Agent.

Application – For manual or automated use.

Instructions For Use

Important: Leave to settle for 8 hours prior to use.

1. Use only in a well ventilated area.
2. Slides should be well drained of clearing agent prior to mounting.
3. Apply mountant and coverslip to slide.
4. Allow the slides to dry in a horizontal position for 1 hour prior to moving or viewing on a microscope.
5. Allow the mounted slides to dry for 48 hours (DPX and ExPert) and 72 hours (ExPert XTF) prior to filing.

NOTE: The use of excessive heat to dry the slides is not recommended.

6. Dispose of empty bottle in line with government guidelines.

NOTE: The coverslip and DPX, ExPert or ExPert XTF mounting medium can be removed by soaking a mounted slide in the clearing agent.

Any complaints or adverse incidents need to be reported to CellPath immediately.

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