# System III Cassette

EAI-05, EAI-06, EAI-07

Issue Date: JULY 2023 01.280.045 V6

CellPath Ltd  
80 Mochdre Enterprise Park,  
Newtown, Powys, SY16 4LE, UK  
T: +44 (0)1686 611333  
E: sales@cellpath.com  
cellpath.com

# System III Cassette

## EAI-05, EAI-06, EAI-07

A small device intended to be used in the laboratory to support/contain clinical tissue specimens (e.g., biopsy samples) to facilitate specimen processing [e.g., fixation (encapsulation in paraffin wax), dehydration, infiltration] in preparation for subsequent cytological or histological examination, and storage. It is in the form of a synthetic polymer container, with or without a lid, with drainage holes to maximize fluid exposure during tissue submersion in a parent device into which the container is loaded. This is a single-use device.

For Laboratory Use.

Store in a dry place. Don't store in direct sunlight.

This product does not expire under normal environmental conditions.

Marking capabilities: Leica IPC, Sakura Autowrite, Primera Signature, DiaPath AP Itineris, CellMark pens.

Primera Signature Printing recommendations – Heat Setting 100 – 115. Ensure print template is set to rounded corners.

Processing Limitations: Suitable for high temperature processing. Suitable for microwave processing.

## Instructions for Use

1. Insert the thread into the cassette printer. Holding the top cassette pull the thread up to release.

1. Insert cassettes manually into printer loading system.

2. Place tissue into the System III Cassette. We recommend that only one sample grossed between 2mm and 4.5mm thick is placed in the cassette. Please ensure tissue is >1.1mm for slotted cassettes, >0.90mm for biopsy cassette and >0.38mm for micro biopsy cassettes.

3. Close the lid and place the cassette in a fixative ensuring it is fully submerged.

4. Process the cassette through solvents according to your usual protocol until the specimens are ready for embedding. If you are using any unusual reagents, we suggest that you

carry out preliminary chemical compatibility tests before using System III Cassette to contain tissue samples.

4. Carefully open the cassette.

5. Embed, cool and section as per your usual protocol.

6. Dispose of the lid in accordance with laboratory guidelines.

**NOTE:** The use of a block trimmer, temperature 85 degrees, may have an effect upon printed text and barcodes therefore, care should be taken to avoid movement when dewaxing the printed face.

**NOTE:** Do not use a knife to scrape excess wax off the printed face. Do not pick excess wax off the printed face.

DO NOT REUSE THIS PRODUCT.

**Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.**

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