Trimming Blades

Trimming Blade Handle

CAH-0100-00A, CAH-0200-00A, CAH-0300-00A, CAH-0400-00A

Issue Date: FEB 2023 01.415.560 V4

EMERGO EUROPE
Westervoortsedijk 60,
6827 AT Arnhem
The Netherlands

CellPath Ltd
80 Mochdre Enterprise Park,
Newtown, Powys, SY16 4LE, UK
T: +44 (0)1686 611333
E: sales@cellpath.com
cellpath.com

Trimming Blades

Trimming Blade Handle

CAH-0100-00A, CAH-0200-00A, CAH-0300-00A, CAH-0400-00A

A flat instrument with a sharp edge intended to be fixed onto a handle (not included) to create a knife for cutting and/or slicing human tissues in a histopathology laboratory for subsequent in vitro diagnostic investigation. It is typically made of stainless steel. This is a reusable device.

A hand-held device upon which a blade (not included) is intended to be mounted and secured to create a knife designed to manually cut and/or slice human tissues in a histopathology laboratory for subsequent in vitro diagnostic investigation. It is typically made of synthetic polymer (e.g., plastic) or wooden material. This is a reusable device.

For Laboratory Use.

Store in a dry place.

This product does not expire under normal environmental conditions.

WARNING: Product is sharp.

Instructions for Use

1. Unscrew locking screw all the way.
2. Carefully slide blade in from the top until the hole in the blade, and the blade screw align. Screw then needs to be tightened back up by hand.
3. Assembly is now complete.

NOTE: DO NOT RESHARPEN.

NOTE: Ensure product is cleaned adequately prior to reuse.

4. Dispose of in line with government guidelines.

NOTE: USED BLADES CAN BE DISPOSED IN THE BOTTOM OF THE DISPENSER.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

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